CATHETER INTRODUCTION WITH A MICROSELDINGER KIT

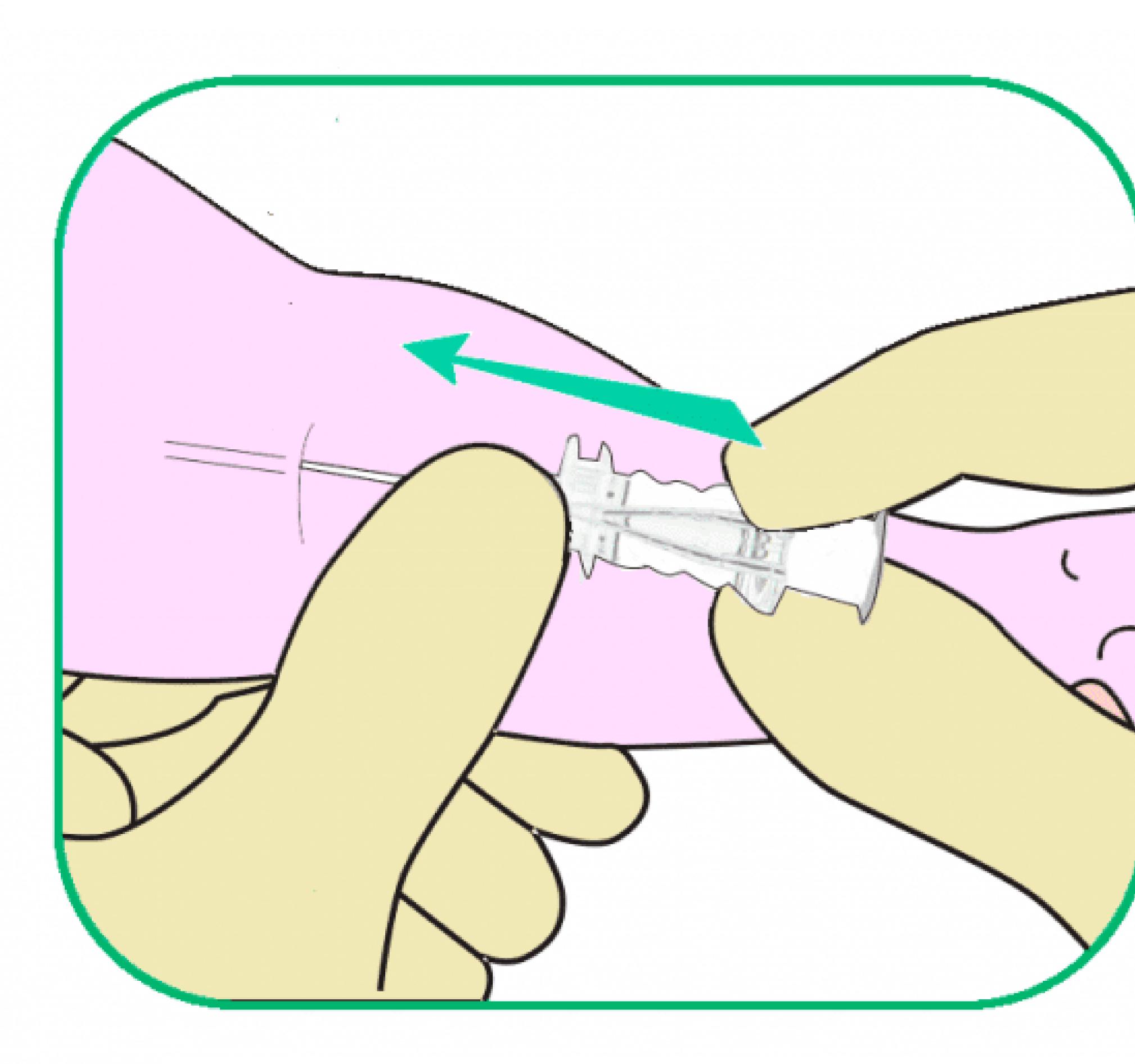


Fig. 1. Puncture with the needle

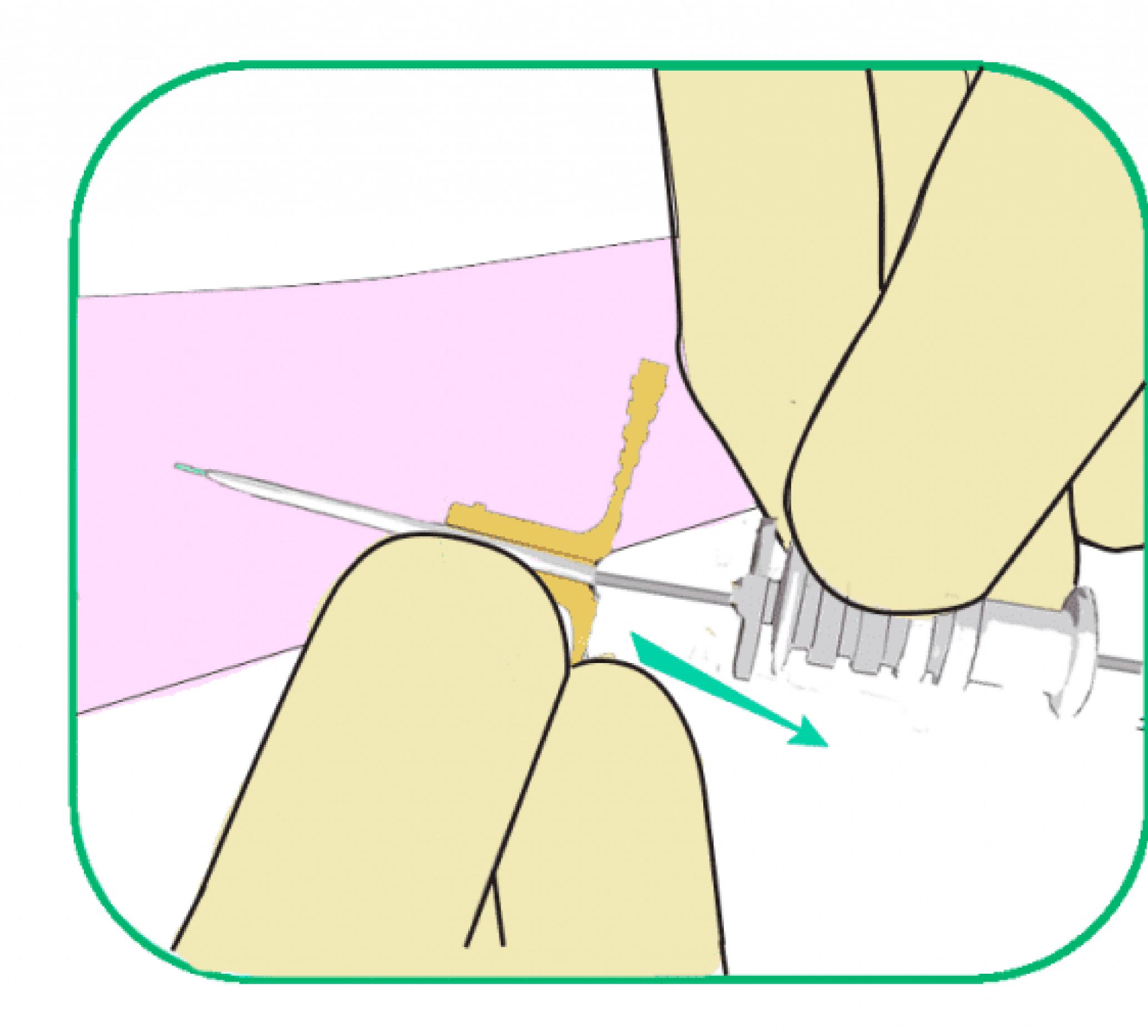


Fig. 5. Withdrawal of the guide and the dilator



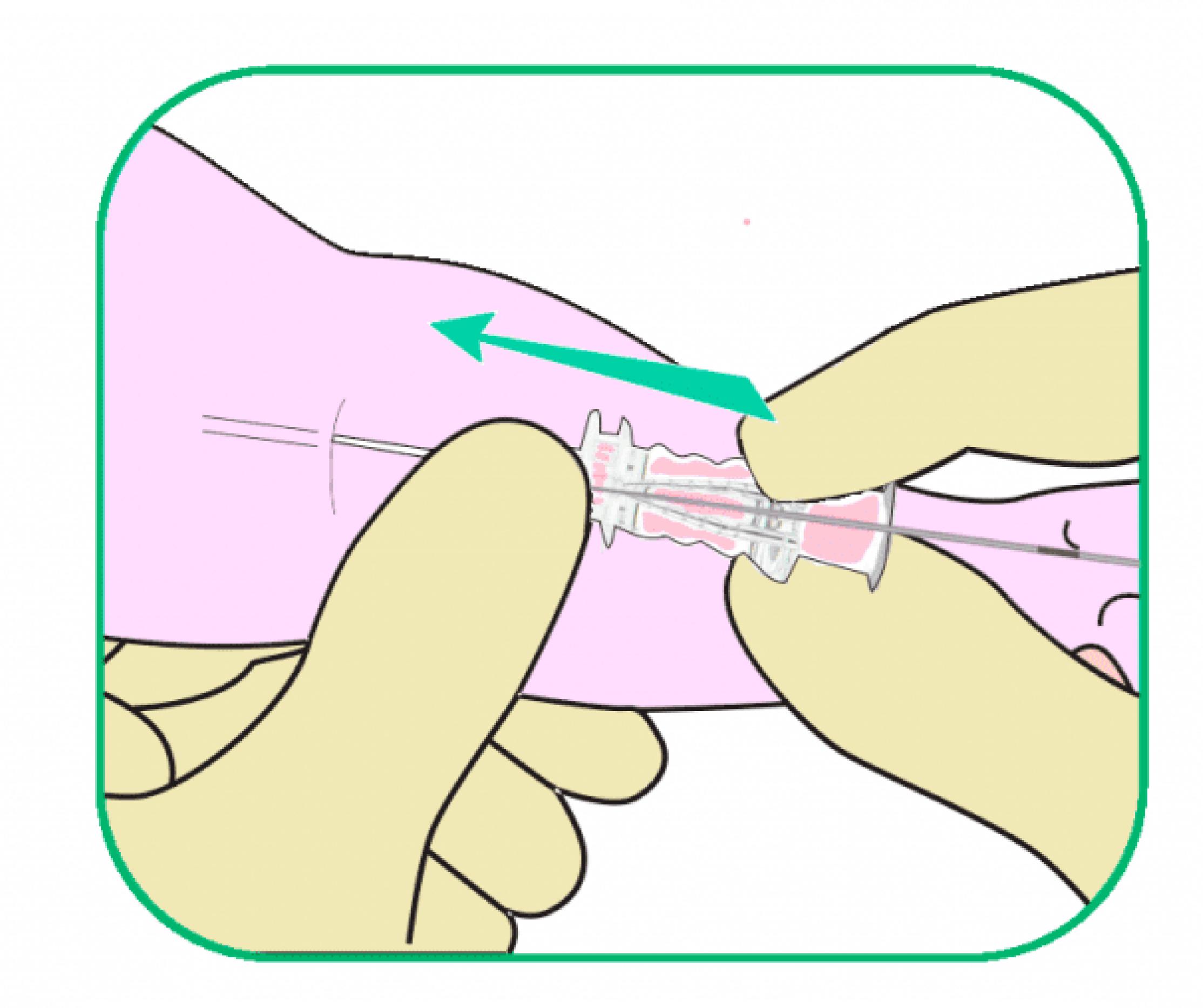
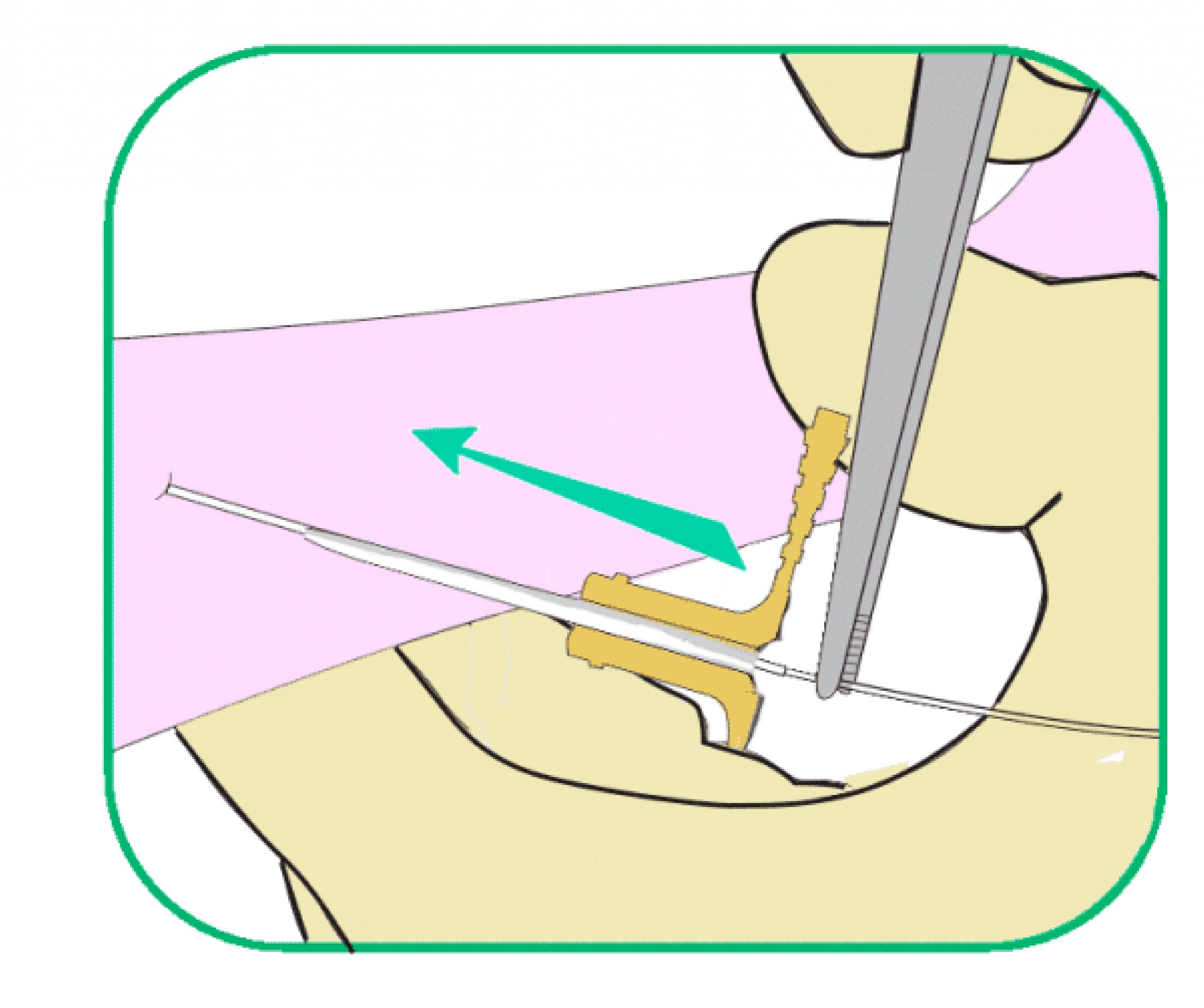


Fig. 2. Insertion of the guide





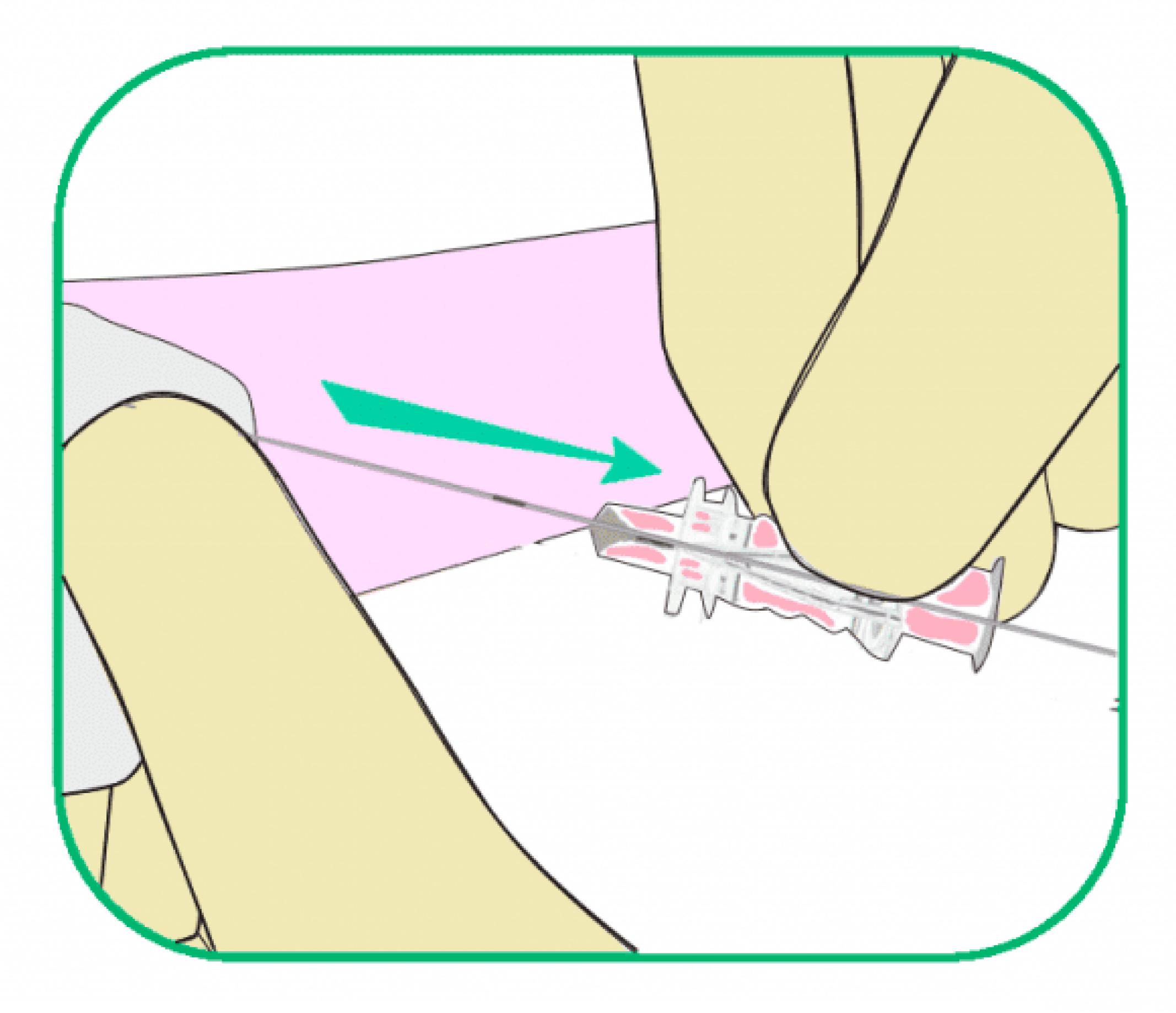


Fig. 3. Withdrawal of the needle

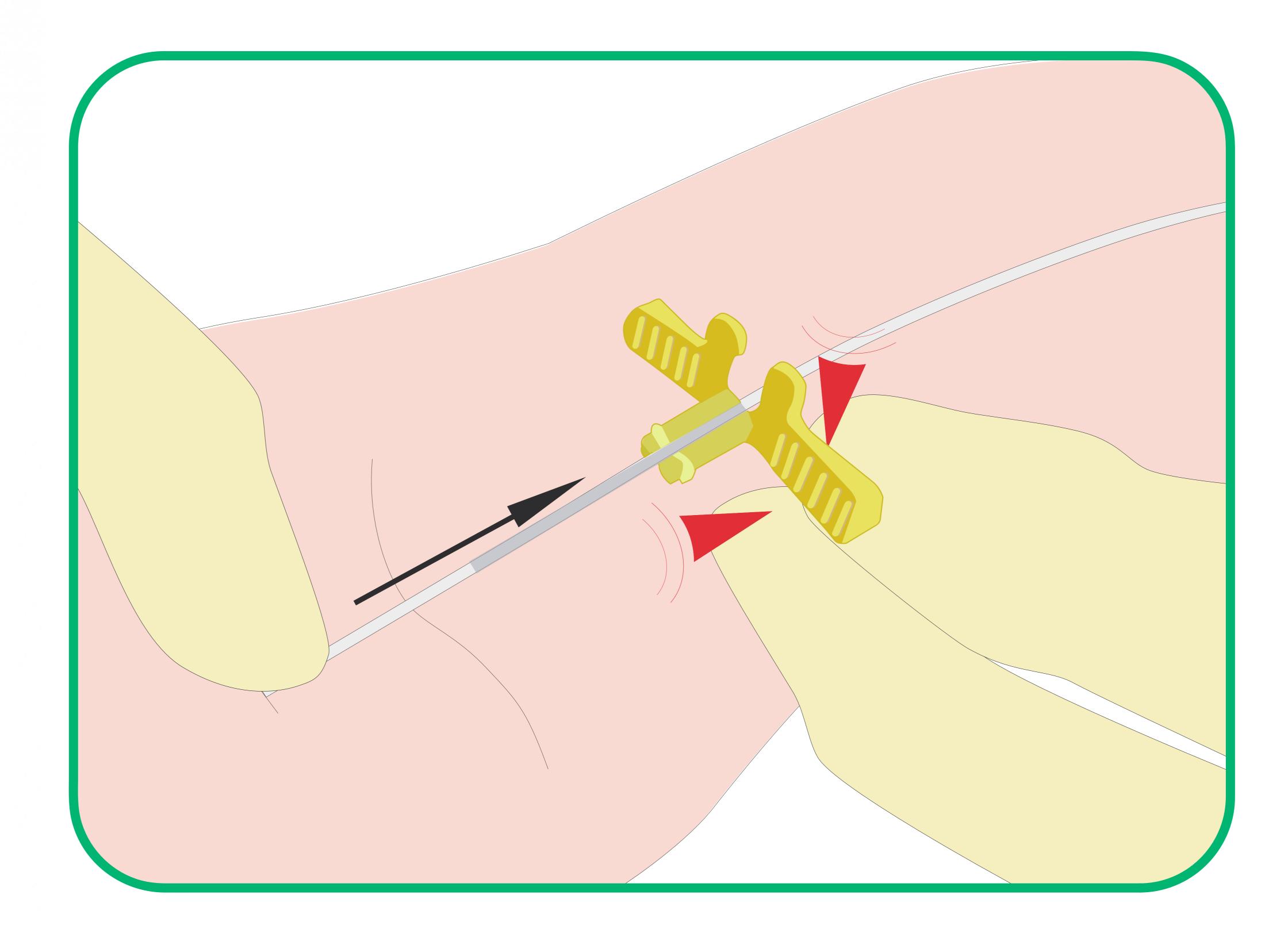


Fig. 7. Secure the catheter by applying gentle pressure with a finger beyond the cannula and slowly remove the cannula.

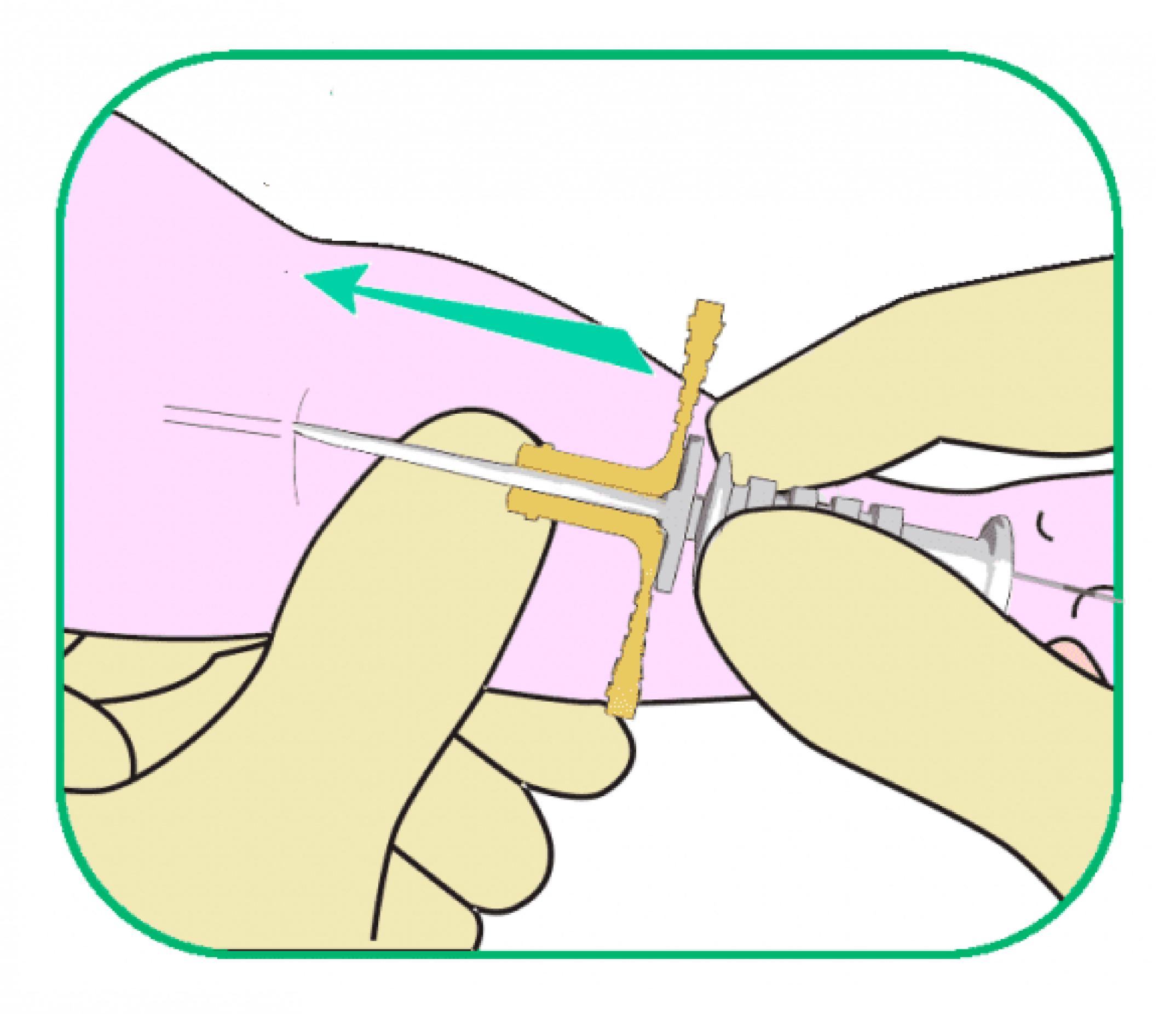


Fig. 4. Insertion of the cannula

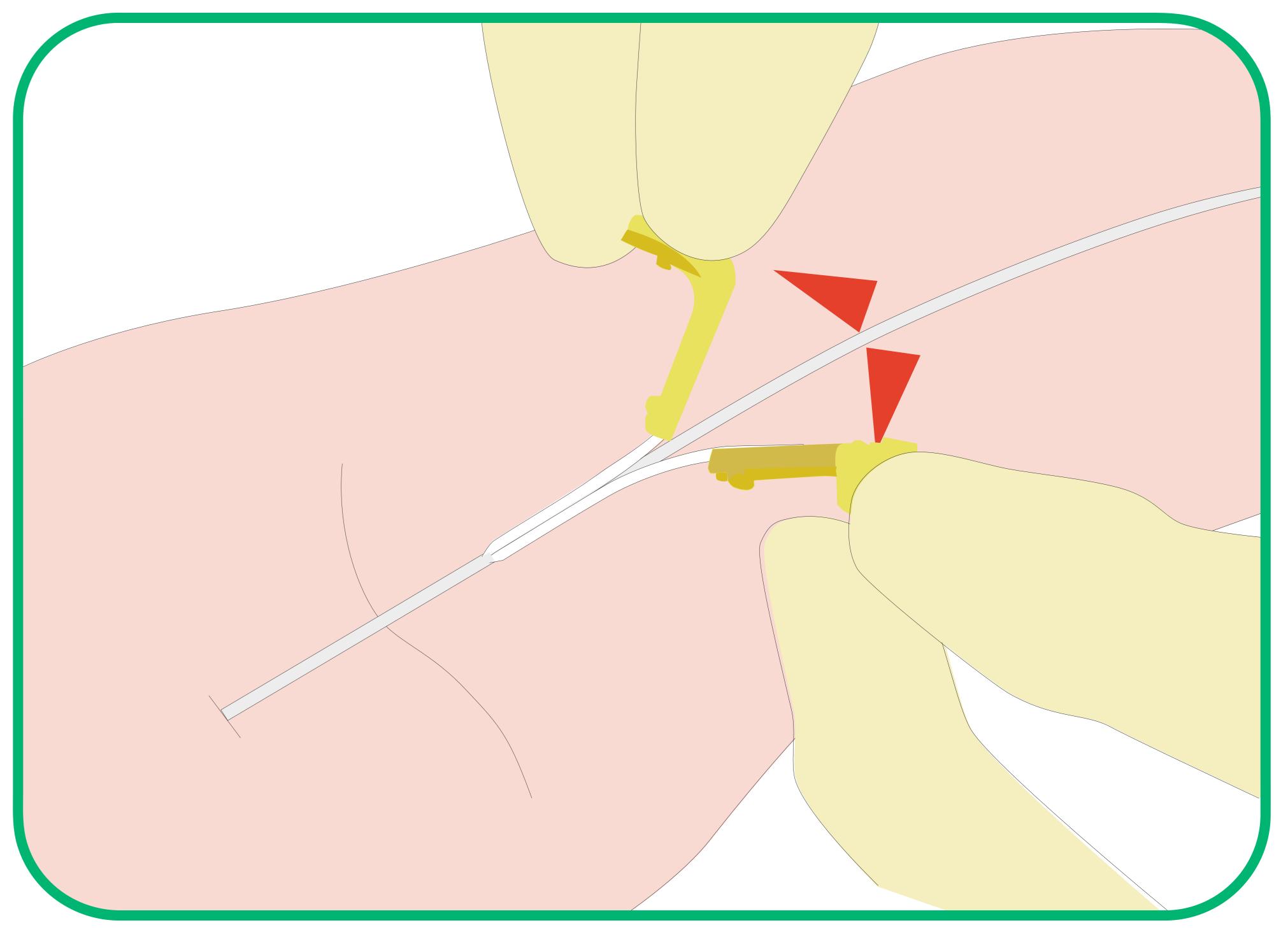


Fig. 8. Once the cannula is outside of the patient's body, slowly peel the cannula by pulling on the wings, being careful not to move the catheter. Finally, advance the catheter to the desired position.