




# MICROSELDINGER TECHNIQUE: SET MICROSELDINGER

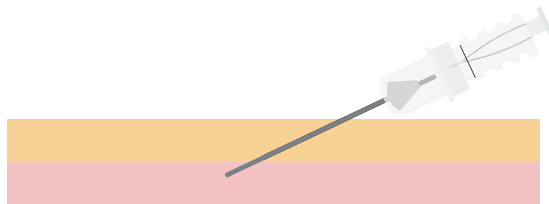


1 Needle puncture

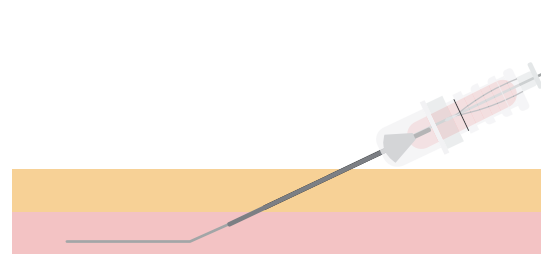


2 Inserting the guidewire through the needle cannula

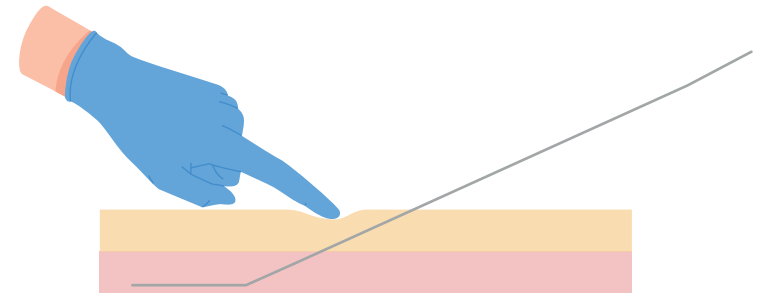


3 Remove the needle while keeping the guidewire inside the vessel

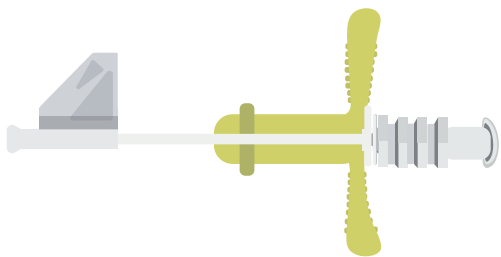


4 Insertion of the insertion aid into the guideway

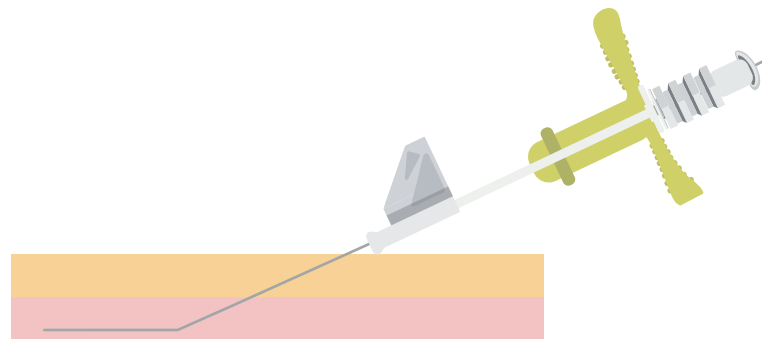


5 Advance the dilator with the aid through the guidewire

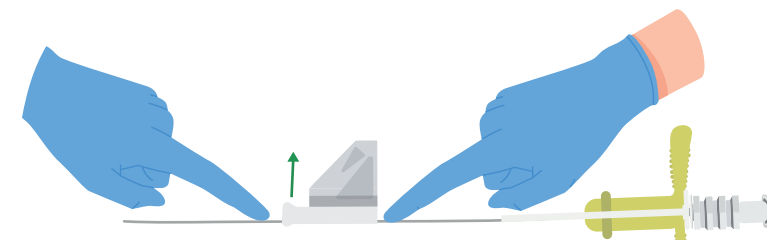


6 Remove the aid and continue dilator insertion

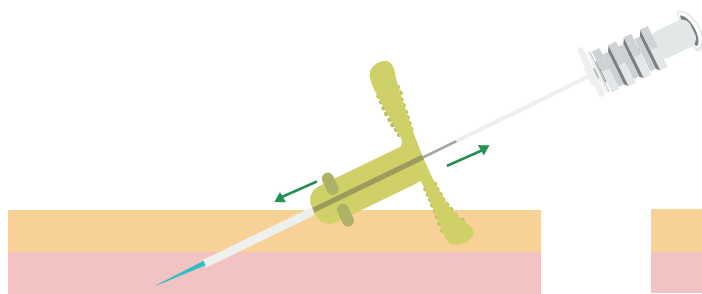

7 Remove the dilator together with the nitinol guide

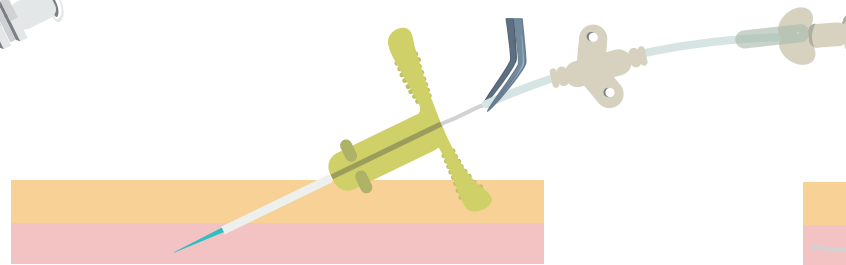

8 Thread the catheter using non-toothed forceps

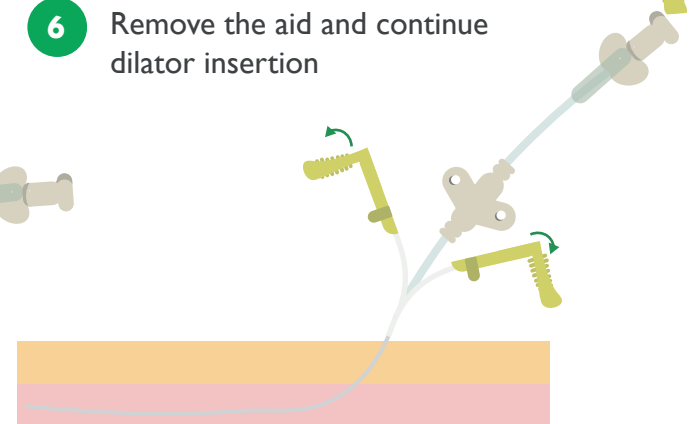

9 Peel-remove the peelable cannula and ensure proper catheter placement