




most-care<sup>Up</sup>

# SET UP IN THREE EASY STEPS

## Step one:

Connect Mostcare Up

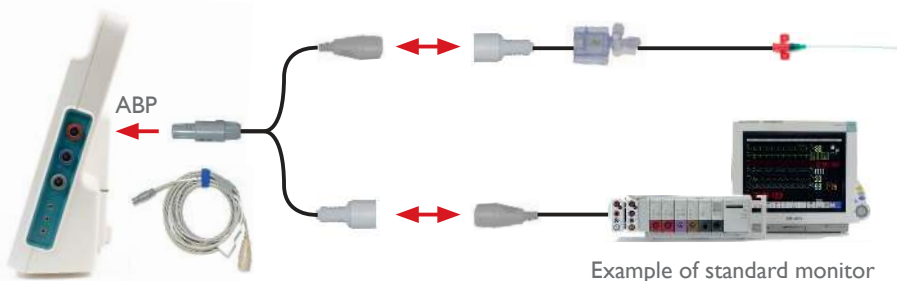


Example of standard monitor

Connect the Y cable between the bedside monitor and the ABP transducer

or

Connect the analogue cable between Mostcare Up and the bedside monitor

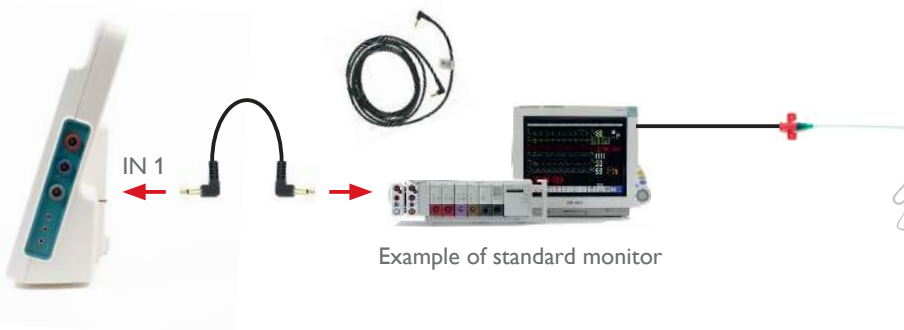

Example of standard monitor

## Scan with your smartphone

to view online or visit [vygon.co.uk/mostcareupevaluation/](https://vygon.co.uk/mostcareupevaluation/)

Tel: 01793 748800 Copyright Vygon (UK) Ltd 2022 Content correct as of: 01/2022 Code: V00612 v5



## Step two:

Enter patient data


1. Press **New monitoring**
2. Enter patient weight and height
3. Select ABP transducer and tick the **Y cable** box if applicable
4. Check other settings and press **Begin monitoring**



## Step three:

Zero the transducer

- Place the transducer at the level of the phlebostatic axis
- Open the transducer stopcock to measure the atmospheric pressure
- Touch the curve on the screen and press **Zero ABP**
- Zero the primary monitor and re-establish trace

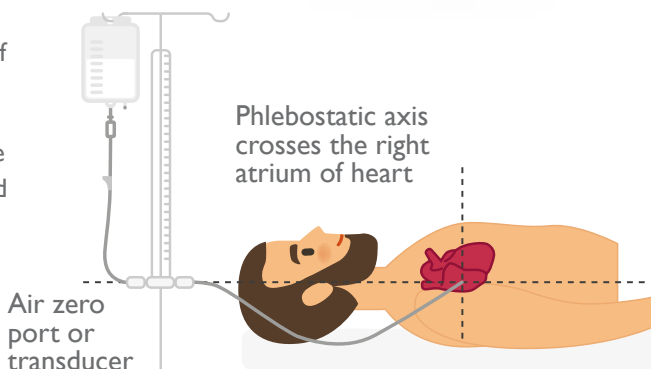

Scan with your smartphone

to view online or visit [vygon.co.uk/mostcareupevaluation/](http://vygon.co.uk/mostcareupevaluation/)

Tel: 01793 748800 Copyright Vygon (UK) Ltd 2022 Content correct as of: 01/2022 Code: V00612 v5

