


PROCEDURE FOR CVC INSERTION


1. Clean the insertion site, according to hospital policy




While using the ultrasound probe, puncture the vein with the needle

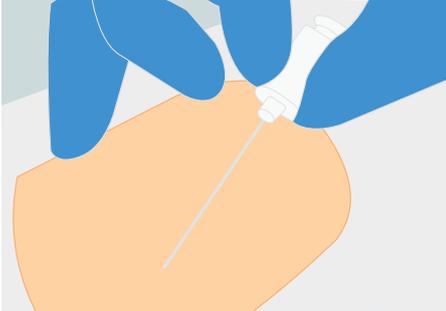

2. Once flashback is visualised, insert the guidewire

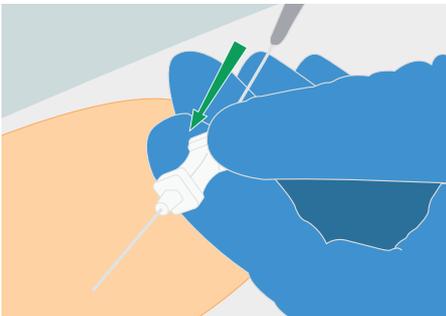


3. Remove the needle

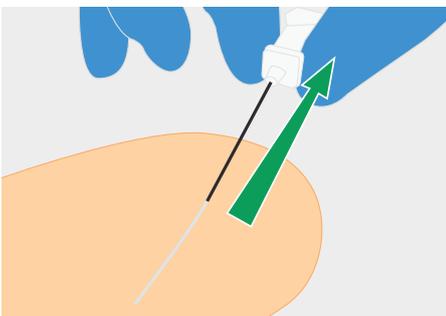


4. Make a small incision with the scalpel




5. Insert the dilator

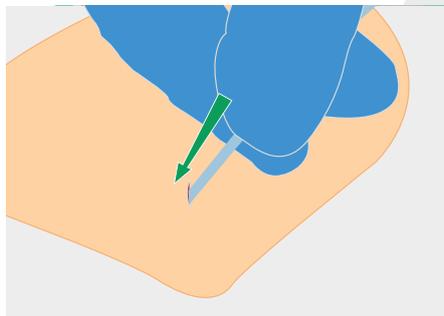


6. Remove the dilator carefully

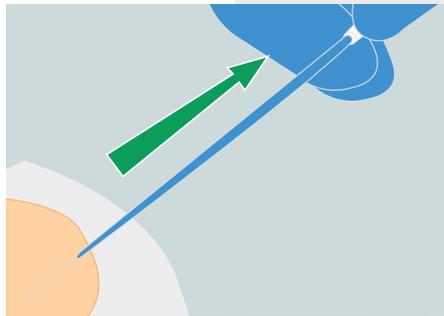


7. Insert the catheter over the guidewire

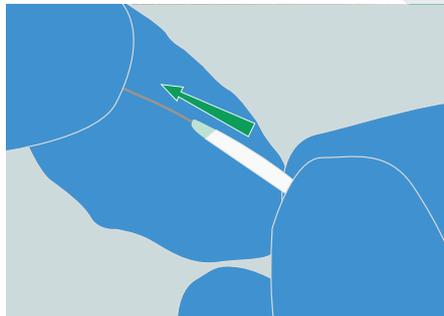

8. Insert the catheter to the hub

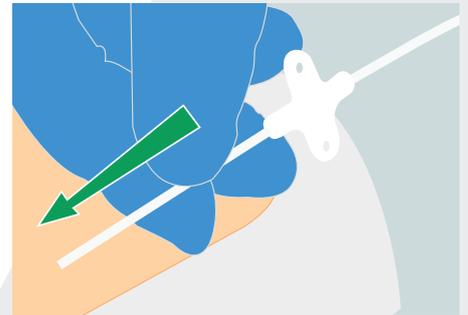

9. Remove the guidewire

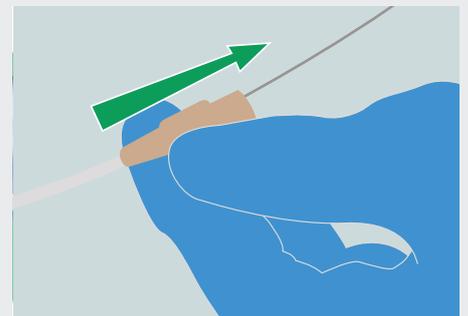

10. Secure the catheter using sutures or an alternative securement device, such as a subcutaneous anchor securement system, following local hospital policy.

