

Fig. 1. Select the vein to puncture. Place the baby in a convenient and comfortable position. Prepare the puncture site. Place sterile fields and apply a tourniquet.

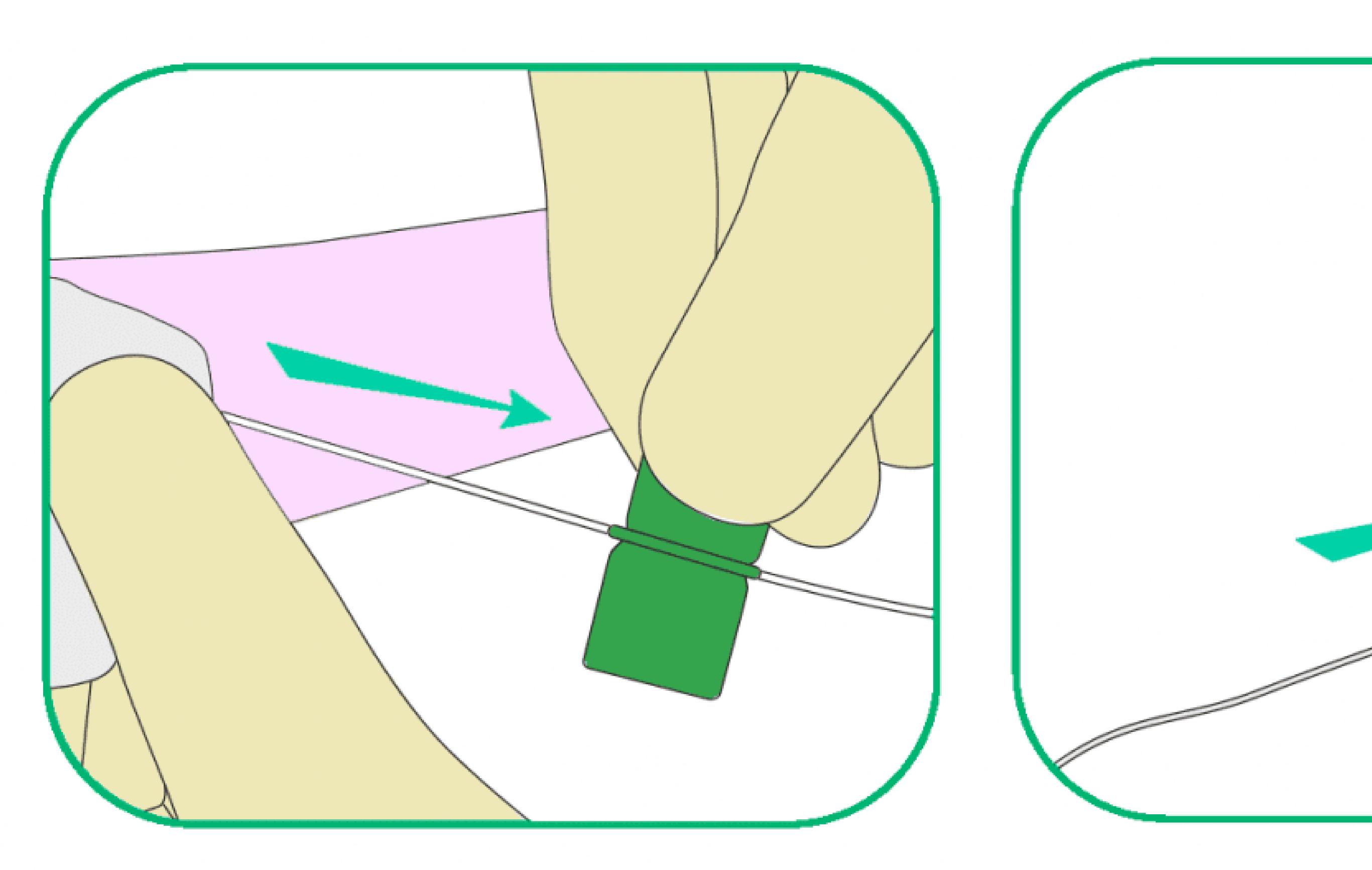


Fig. 4. Apply slight pressure with a finger beyond the needle and carefully remove the introducer from the vein.





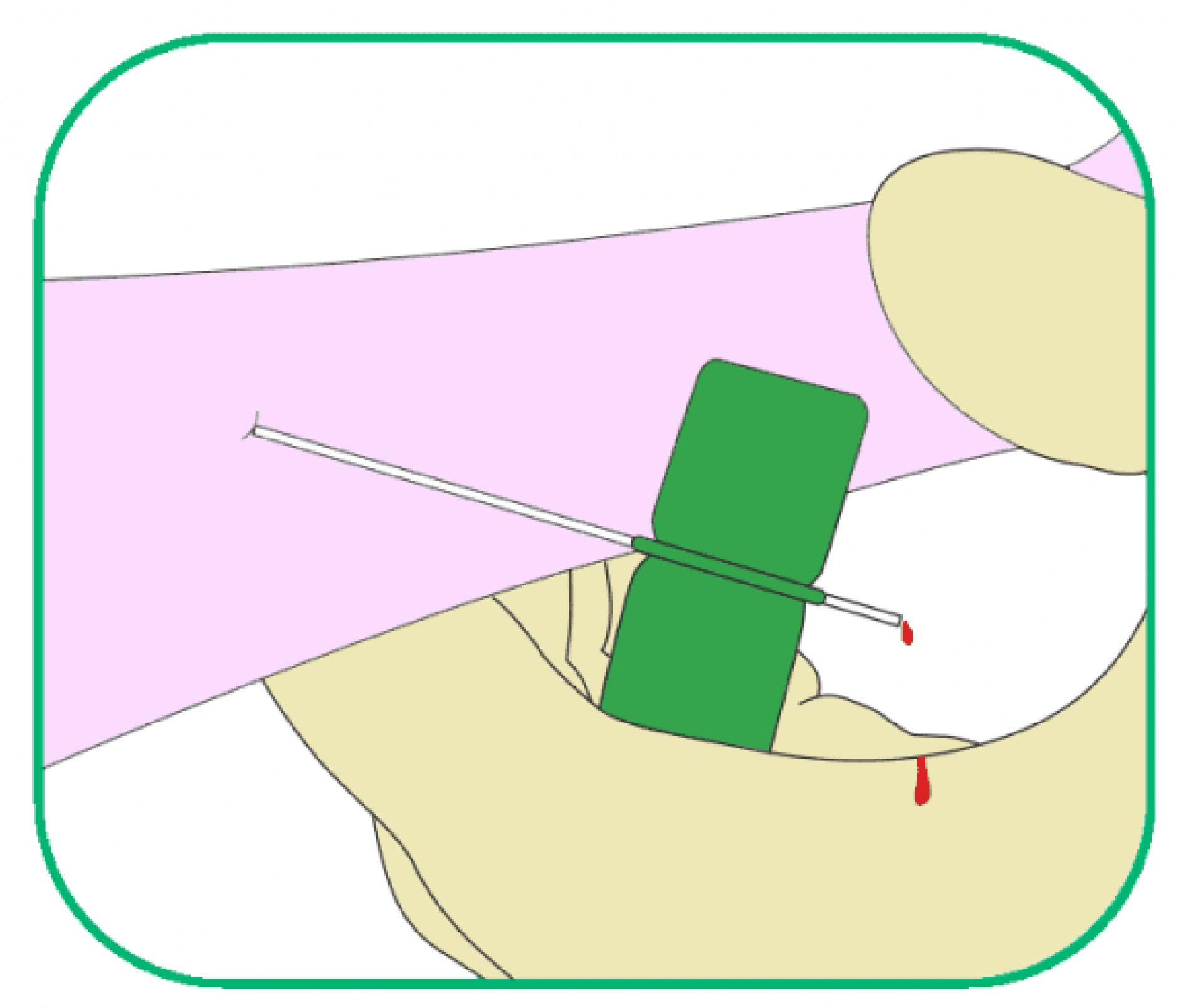
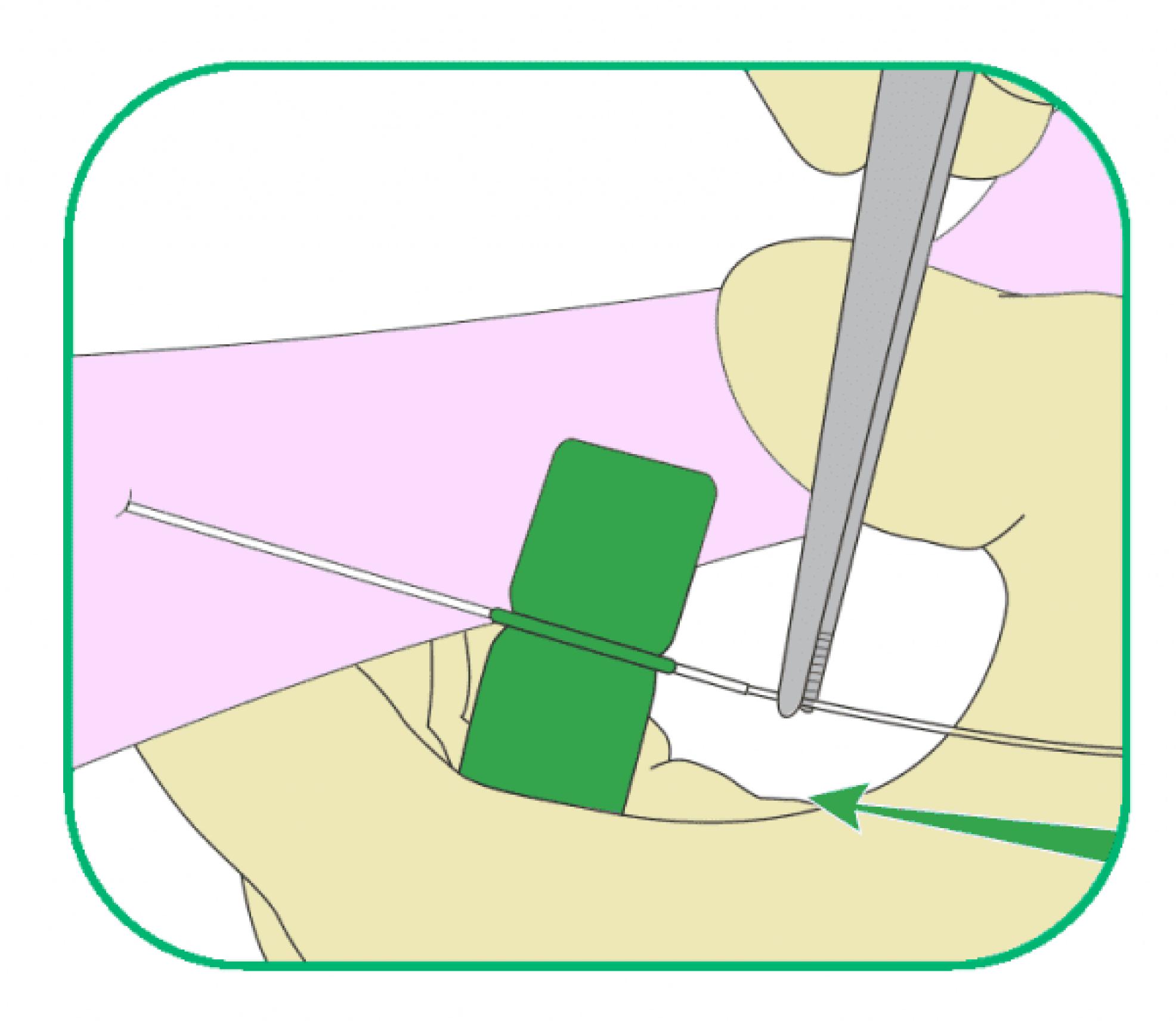


Fig. 2. Perform the venous puncture with the supplied 19G butterfly needle to obtain blood return.

remove the needle.





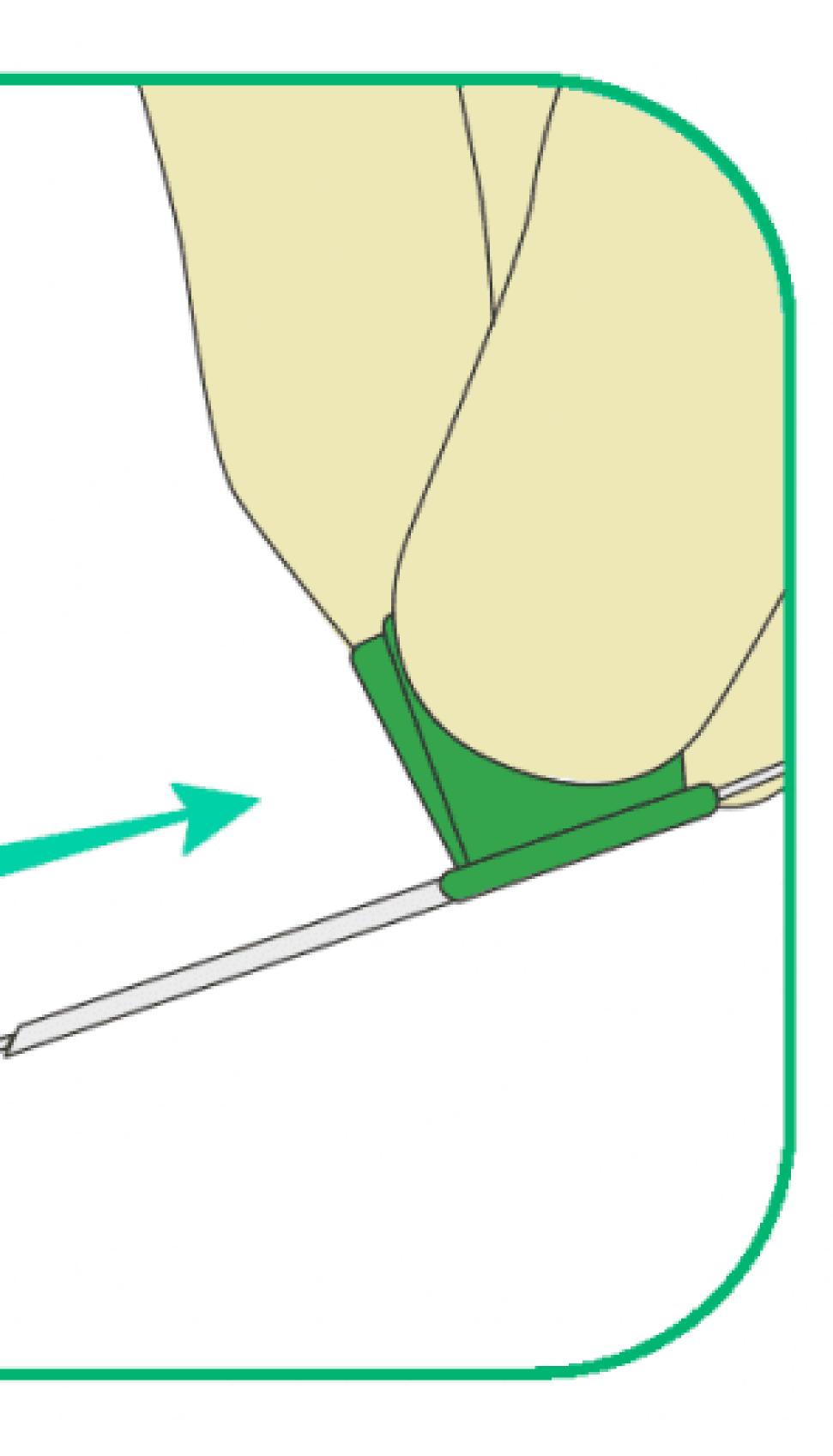


Fig. 5. Keep the catheter straight and carefully

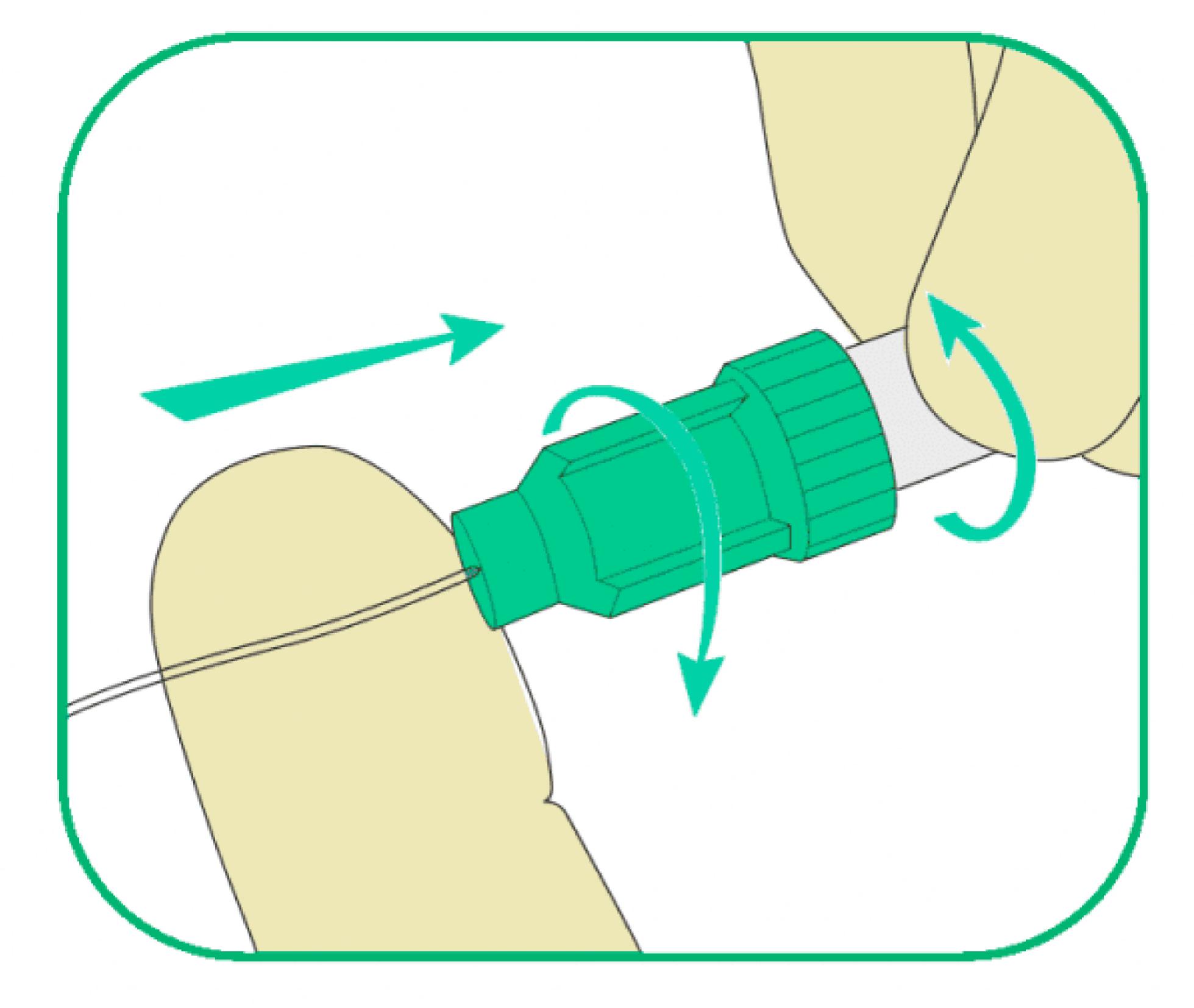


Fig. 6. Loosen the Easy-lock connector to remove the catheter from the easy-lock hub. Do not completely separate the connector from the extension line. Remove the needle, and then replace the proximal tip of the catheter into the

easy-lock until the black mark completely disappears inside the connector.

Fig. 3. Pass the catheter through the needle using non-toothed forceps. Insert the catheter enough to release the tourniquet and advance it gradually.