CATHETER INTRODUCTION WITH A PEELABLE CANNULA

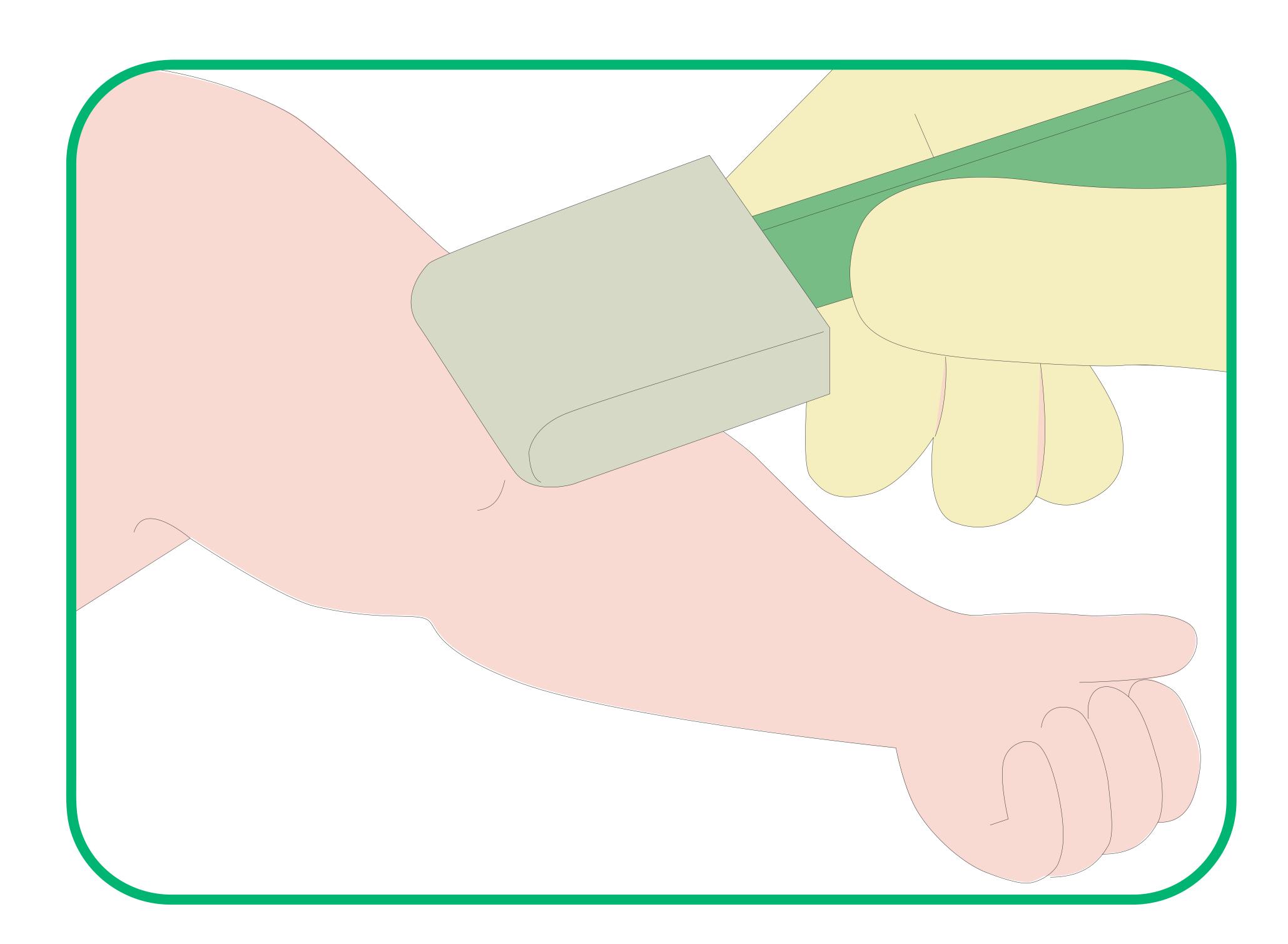


Fig. 1. Select the vein, prepare, clean the skin, and place sterile fields leaving only the puncture site exposed. Apply a tourniquet.

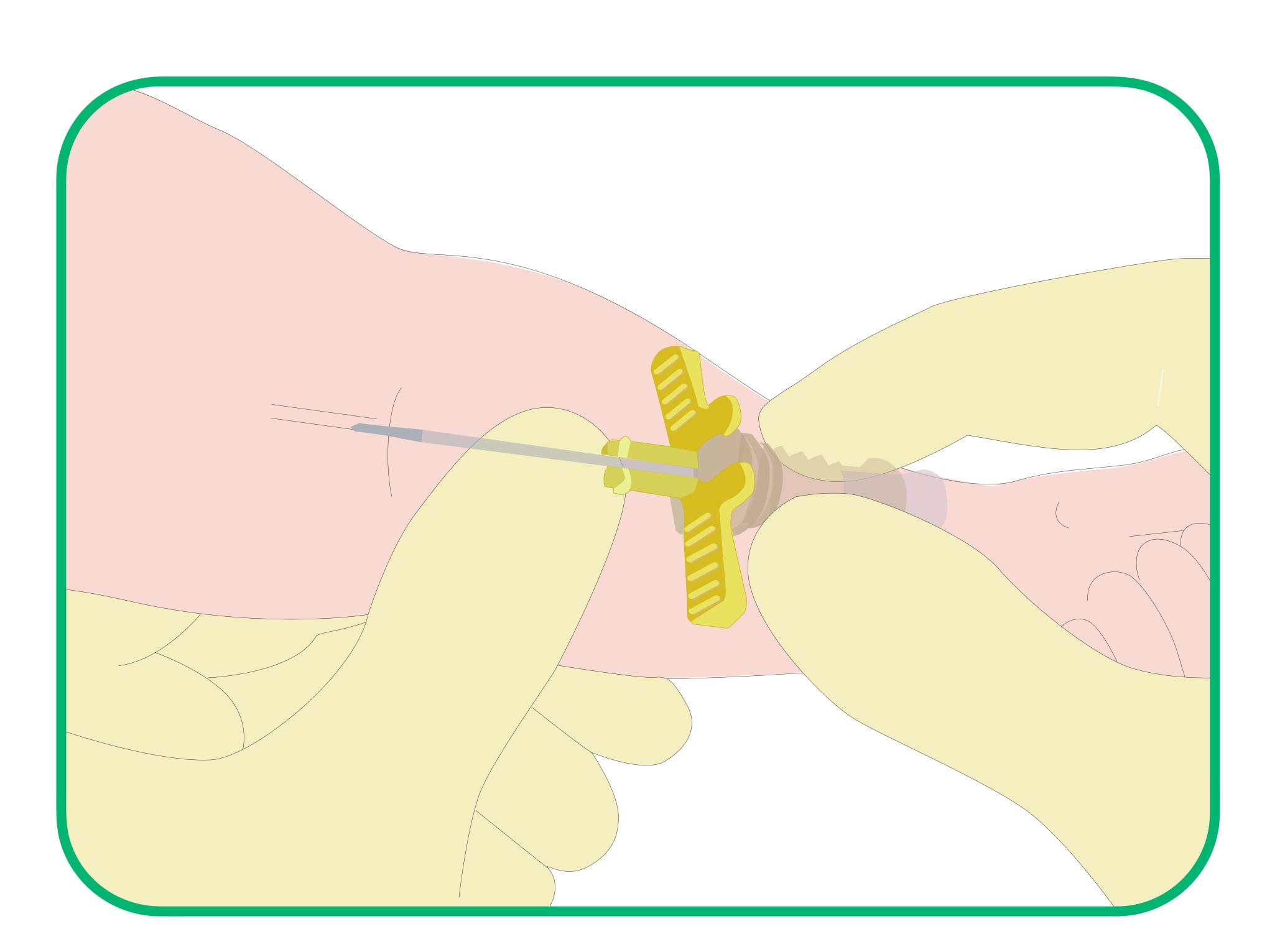


Fig. 2. Perform the venous puncture with the provided microflash peelable cannula at an angle of 15 to 30 degrees. Advance the cannula until blood reflux is observed.

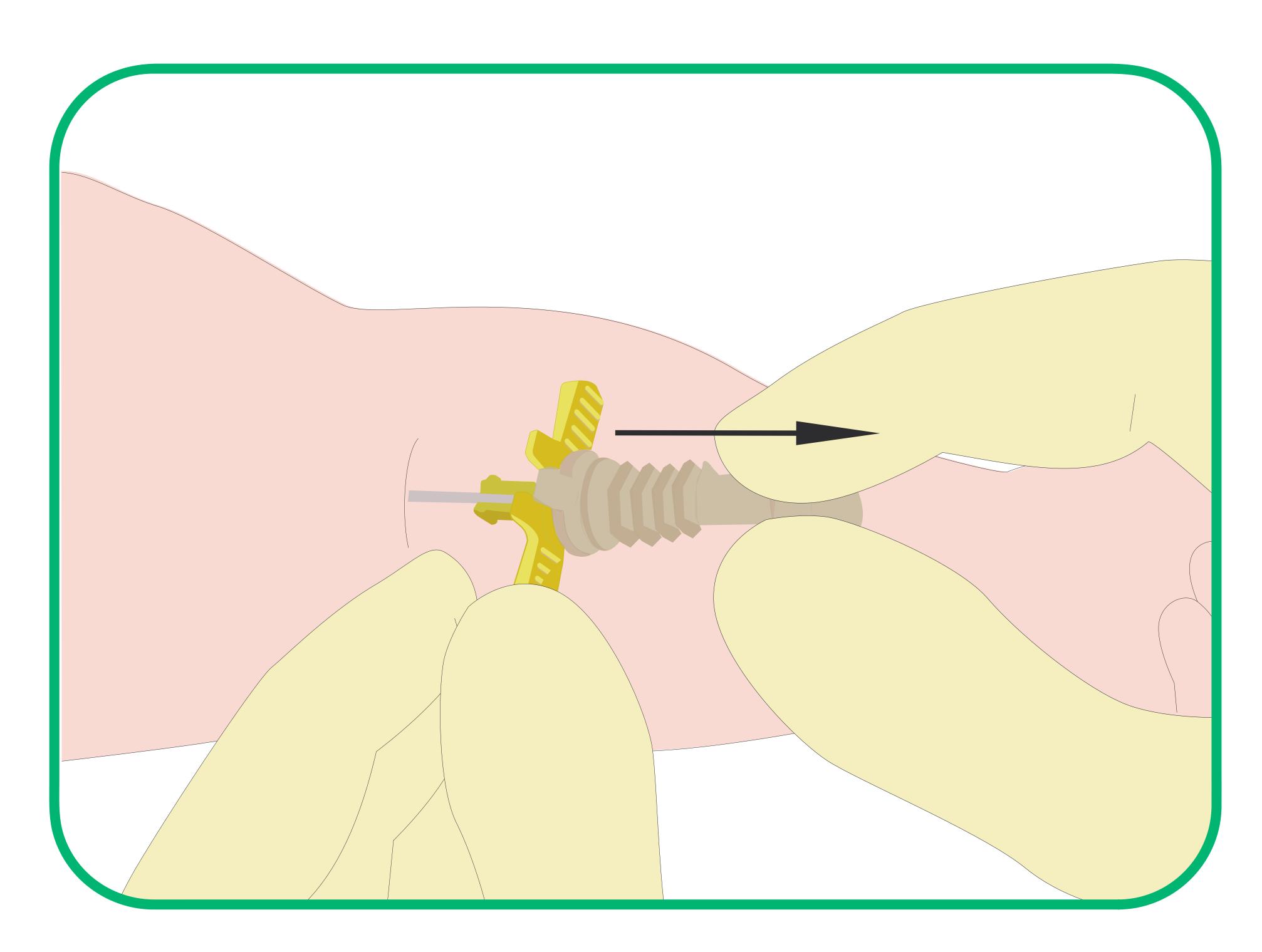


Fig. 3. Lower the microflash a few degrees and advance only the plastic cannula. Remove the tourniquet.

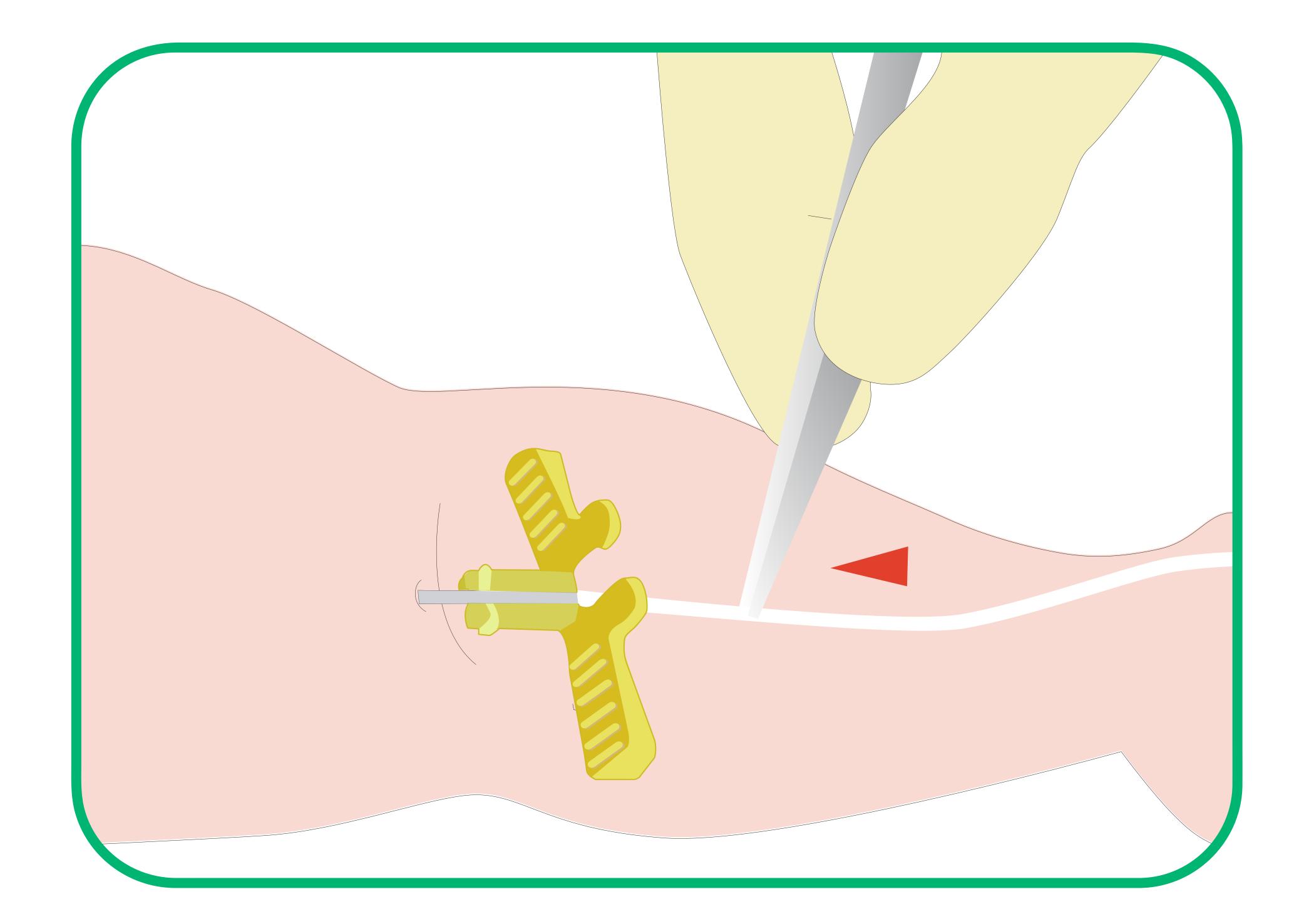


Fig. 4. Pass the catheter through the cannula using non-toothed forceps. Insert the catheter enough to release the tourniquet and advance it gradually.

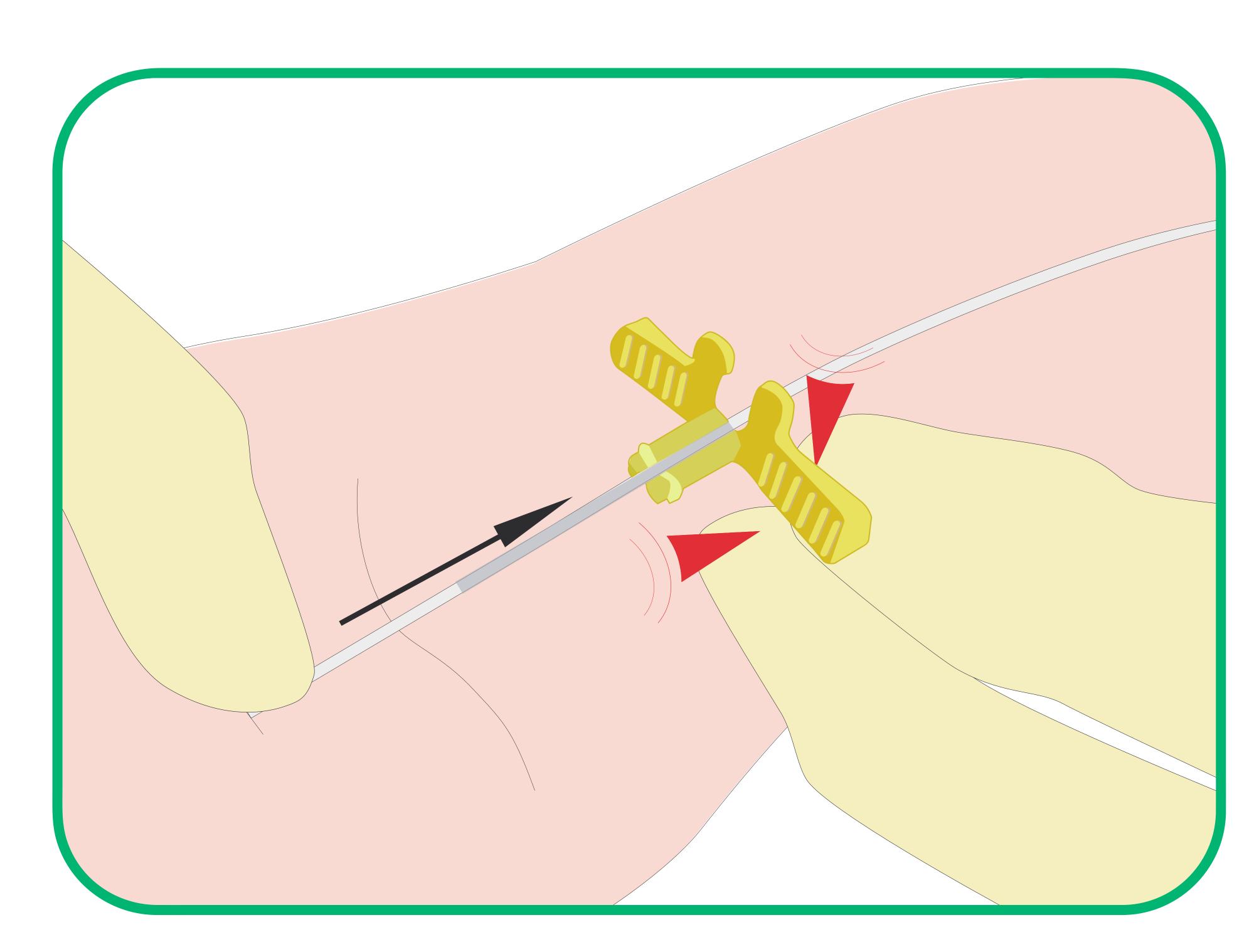


Fig. 5. Secure the catheter by applying gentle pressure with a finger beyond the cannula and slowly remove the cannula.

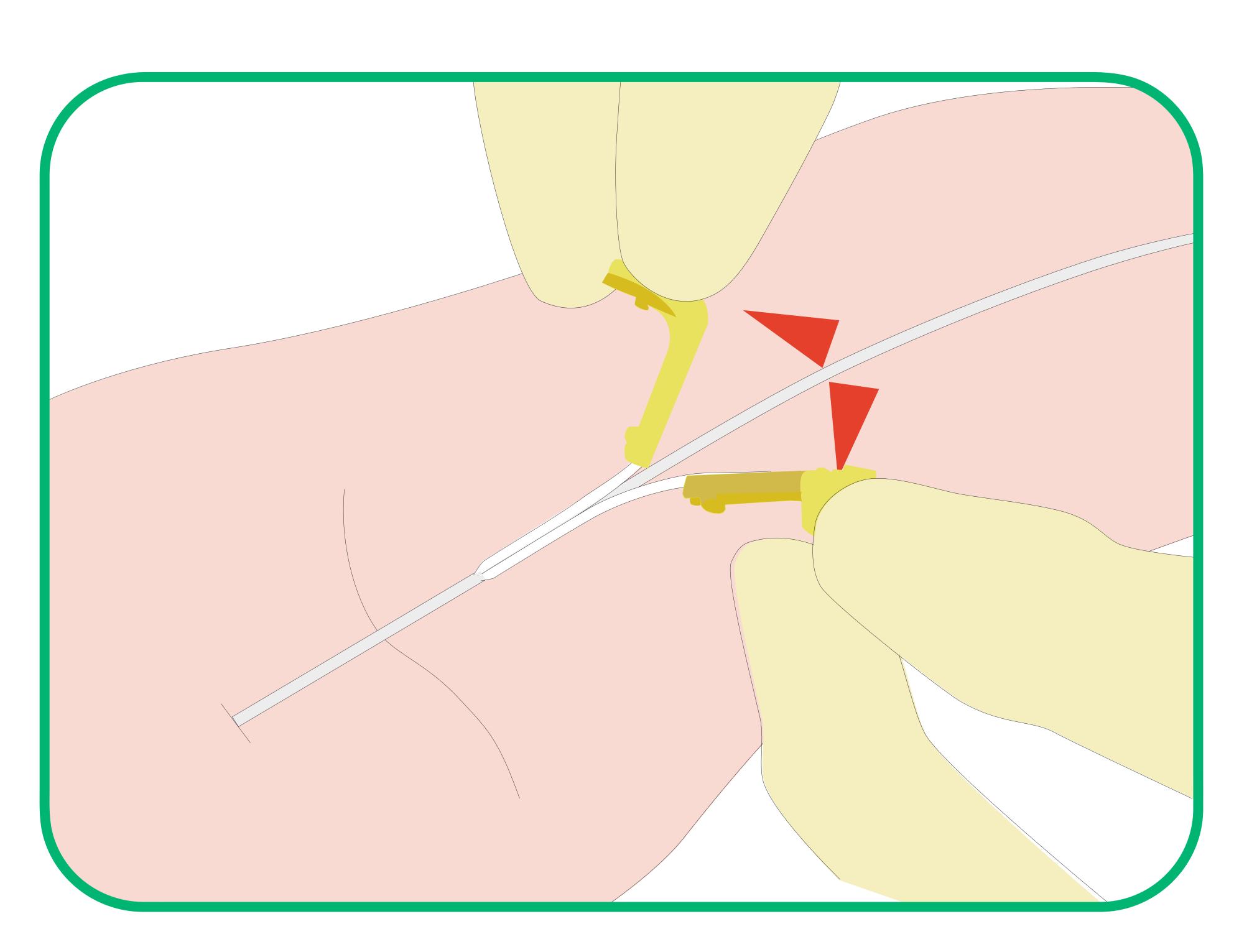


Fig. 6. Once the cannula is outside of the patient's body, slowly peel the cannula by pulling on the wings, being careful not to move the catheter. Finally, advance the catheter to the desired position.

